

# Verruca Freeze®

## Cleaning Your CryoCones®



### Disinfecting CryoCones®

According to CDC guidelines, CryoCones® are classified as a non-critical medical device and may be disinfected using an EPA registered intermediate-level or low-level disinfectant. CaviCide® or CaviWipes™ by Metrex are highly recommended. **Please follow the directions provided by the disinfectant manufacturer.**

Cones can be autoclaved on the plastics setting if desired. Refer to autoclave instructions for plastics specifications. **Caution:** Autoclaving CryoCones® may reduce their shelf life.

#### Other Resources

<http://www.metrex.com/surface>

[www.CDC.gov](http://www.CDC.gov)



**CRYO SURGERY, INC.**  
5829 Old Harding Pike  
Nashville, Tn 37205  
p. 615.354.0414  
f. 615.354.0466  
[info@cryosurgeryinc.com](mailto:info@cryosurgeryinc.com)