

VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery
www.verrucafreeze.com



Verruca-Freeze treats all types of warts including the following:

COMMONLY TREATED Warts

Common Warts (Verruca vulgaris)

Tips: Easy to treat, one saturated CryoBud can treat multiple lesions.



**Freeze Time:
30 - 40 Seconds**

Flat Warts

Tips: Most often, these lesions affect the hands or face, and a linear arrangement is not uncommon.



**Freeze Time:
20 - 30 Seconds**

Filiform Warts (Digitate wart)

Tips: Use tweezers to pull the lesions away from the skin and freeze the base of the lesions.



**Freeze Time:
30 - 40 Seconds**

Plantar Warts

Tips: Using the freeze thaw freeze procedure can help achieve maximum resolution.



**Freeze Time:
40 Seconds**

CPT Code:

17110 (1-14)
17111 (>15)

Reimbursement:

\$89.00 - \$108.38
\$84.94 - \$129.11

Allow a thaw time of approximately 40 seconds. We recommend freeze-thaw-freeze on lesions greater than 5mm and plantar warts.



VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery
www.verrucafreeze.com



Did you know Verruca-Freeze treats 21 different types of skin lesions?

COMMONLY TREATED LESIONS

Skin Tags (Acrochordons)

Tips: Often irritating but easy to treat, skin tags form primarily in areas where the skin forms creases, such as the neck, armpit and groin.



Freeze Time:
25 - 30 Seconds

CPT Code
11200 (1-15)
Reimbursement
\$83.58 - \$77.39

Molluscum Contagiosum

Tips: This is a common pediatric condition. One saturated CryoBud can treat multiple lesions. These lesions are often 1-5 millimeters in diameter, with a dimpled center.




Freeze Time:
20 Seconds

CPT Code
17110 (1-14)
Reimbursement
\$108.38 - \$89.00

Seborrheic Keratoses

Tips: Seborrheic keratosis is also known as Senile warts. It is seen more often in middle age patients, found in groups on the patients back.

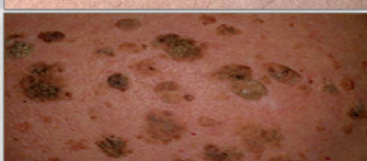

Freeze Time:
25 - 30 Seconds

CPT Code
17000 (1)
Reimbursement
\$87.00 - \$55.38

Actinic Keratoses (Solar Keratoses)

Tips: Actinic keratosis is very common, affecting half of the global population. It is skin damage caused by years of exposure to ultraviolet rays.



Freeze Time:
20 - 35 Seconds

CPT Code
17000 (1)
Reimbursement
\$87.00 - \$55.38

Age Spots (Lentigo)

Tips: Age Spots are blemishes on the skin associated with aging and exposure to ultraviolet radiation from the sun often found on the hands and is easily treated.



Freeze Time:
20 - 35 Seconds

CPT Code
17000 (1)
Reimbursement
\$87.00 - \$55.38

Genital Warts (Condylomata acuminata)

Tips: Highly contagious sexually transmitted disease caused by some sub-types of human papillomavirus.



Freeze Time:
20 - 30 Seconds

CPT Code
54056
Reimbursement
\$307.00 - \$111.44

