

VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery
www.verrucafreeze.com



Treat animals with warts, skin tags & other benign skin lesions!

Verruca Freeze® is a great alternative to scalpel removal because it is completely bloodless and requires no anesthetic, making it great for older pets. CryoBuds® are great for treating hard to reach areas on paws, ears and faces!



Common Lesions Treated With Verruca Freeze®

- ✓ **No Anesthetic!**
- ✓ **No Scalpel!**
- ✓ **No Sutures!**
- ✓ **One Minute Procedure!**
- ✓ **Flexible Purchasing Options!**


Warts: a firm abnormal elevated blemish on the skin; caused by a virus.

Skin Tags: small benign tumors that form primarily in areas where the skin forms creases, such as the neck, armpit and groin.

Distichia: is an eyelash that arises from an abnormal spot on the eyelid of a dog. Distichiae usually exit from the duct of the meibomian gland at the eyelid margin. They usually form in multiple and sometimes more than one arises from a single duct.

Sarcoids: are common, usually benign, skin tumours on horses and other equidae which have several different forms.

Sebaceous Adenoma: a cutaneous condition characterized by a slow-growing tumour usually presented as a pink, flesh-coloured, or yellow papule or nodule.

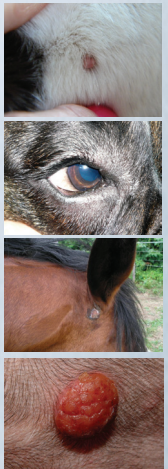

Application

With two forms of application, treating benign skin lesions is now easier than ever! For boney or delicate areas we recommend using the CryoBuds®: simply spray onto a CryoBud® until it drips once then place the bud onto the lesion for 25 seconds. For fleshy areas or large lesions we suggest the CryoCones®: choose the cone that is 1 millimeter greater than the size of the lesion you are treating, place the cone over the lesion (rubber tip down) and spray into the inside wall of the cone for 3-6 seconds, wait for the cryogen to boil off completely, then remove the cone. After the procedure, allow the lesion to thaw for 45 seconds and the process is complete. That's it! The lesion will blister slightly, scab, then fall off within 7 -10 days.

Tips For Treating Animals

Animals generally have thick skin, so debride thick, calloused warts before treatment. Use one freeze-thaw cycle for lesions on thin skin or lesions smaller than 5mm. Use two freeze-thaw cycles during one office visit for lesions 5mm or larger. The 12mm cone can be flipped to treat lesions that are 20-25mm.

